ARNOMED KALT-/WARMPACK

Kalt-/Warmpack zur Therapie Cold/Heat pack for thermotherapy

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Reusable cold and heat pack for external use. Provides local pain relief for swelling and promotes blood circulation and muscle relaxation.

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Cold pack suitable for pain relief and reduction of swelling. Heat pack suitable for promoting circulation and muscle relaxation.

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Application cold pack:

1. Place the cold pack in the freezer for at least 2 hours before use (up to -18 °C).
2. Wrap the cold pack in a cloth before use.
3. Use the cold pack for a maximum of 20 minutes at a time.

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Application heat pack:

1. A)

Art. no. 932-M-2, 932-L-2: Place the heat pack in boiling water for about 3 minutes (up to 95°C).

Art. no. 932-XL-2: Place the heat pack in boiling water for about 4 minutes (up to 95°C).

B)

Art. no. 932-M-2, 932-L-2: Heat the heat pack in the microwave for about 65 seconds (at 600W).

Art. no. 932-XL-2: Heat the heat pack in the microwave for about 2 minutes (at 600W).

2. Let it cool briefly after removal.

3. Wrap the heat pack in a cloth before use.

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For external use only. Inspect the product for damage before use and dispose of leaking packs. Follow the instructions carefully. Not suitable for individuals with sensory impairments. Keep out of reach of children. Discontinue use immediately if discomfort occurs. If the liquid comes into contact with skin or eyes, rinse with water and consult a doctor. Do not open or puncture the pack. After multiple uses, the pack may become unusable. Test the product's temperature cautiously after heating.

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Medical device in accordance with Regulation (EU) 2017/745 (MDR) with CE marking

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Store dry, at room temperature, and protect from direct sunlight.

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4 years from the production date

Letzte Änderungen
Last Changes

17.10.2025